Gebrauchsanweisung
Instructions for Use
Mode d'emploi

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
Gruppe

akkubetrieb

▶ = Info

➔ = Eingabe

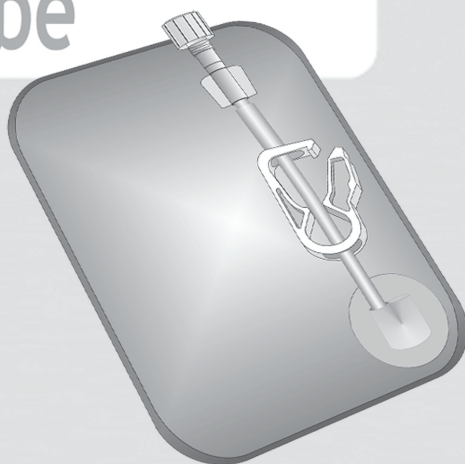PEGA® BAG 50 ml (REF 14050)

PEGA® BAG 100 ml (REF 14100)

PEGA® BAG 150 ml (REF 14150)

PEGA® BAG 300 ml (REF 14300)

PEGA® BAG

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This page left intentionally blank

Intended Use

PEGA® BAG is a medication reservoir equipped with a luer-lock-connection that can be filled with a syringe for parenteral administration of drugs in ambulatory and in-patient infusion therapies. PEGA® BAG is intended for use with the PEGA® Infuse, PEGA® PCA, PEGA® plus and PEGA® light C infusion pumps.

PEGA® BAG is connected to the PEGA® infusion pumps through the peristaltic tube which is part of the products PEGA® SET, PEGA® SET curled and PEGA® COMBI LINE E, S, Y.

Indication for Use

The PEGA® BAG is suitable for intravenous, subcutaneous, epidural, and intrathecal administration of drugs from a PEGA® infusion pump.

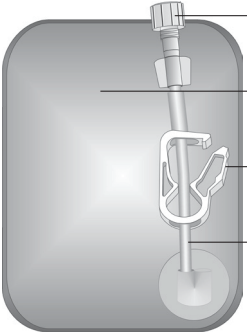
View	No. Description / Specification
	<ul style="list-style-type: none"> <li data-bbox="423 794 770 820">1 Connection LuerLock (female) <li data-bbox="423 868 1005 893">2 Medication reservoir 50 ml, 100 ml, 150 ml, 300 ml <li data-bbox="423 941 649 967">3 Tube clamp (PVC) <li data-bbox="423 1015 904 1040">4 Tube (PVC) Dead volume tube $V_0 = 450 \mu\text{l}$

Fig. A: view PEGA® BAG

English

Contraindication

Do not use drugs which are not compatible with the material (PVC) of the PEGA® BAG.

Additional Information

PEGA® BAG is latex, DEHP and pyrogen free.

General Safety Instructions

- Please consult the manuals of your PEGA® pump before using a PEGA® BAG.
- This disposable item is intended to be used on a patient only once in order to guarantee optimum performance and hygiene.

If the item is used again, even after reconditioning, any liability on the part of Venner Medical (Deutschland) GmbH shall cease to exist.

- Please also respect the manuals of the PEGA® COMBI LINE and PEGA® TUBE.
- Always replace the PEGA® BAG and PEGA® TUBE together.
- Carry out all of these steps on a clean and low germ concentration surface.
- Hands must be disinfected previously. Do not touch open luer-locks (danger of contamination).

- Use only original PEGA® accessories.
- PEGA® BAGs are provided in sterile packages. Do not use any disposable if the package appears damaged or opened.
- Use PEGA® BAGs only once and discard the disposable as per local regulations.
- Only remove the vented caps when filling and connecting to the PEGA® TUBE.
- Do not exceed the indicated maximum volume for the PEGA® BAG.
- PEGA® BAG must not be refilled and not be used longer than 30 days.
- For storing filled PEGA® BAGs use closing caps (not included).
- The pump with the PEGA® BAG must not be positioned higher than 1 m above the heart of the patient during the therapy.

Confirmation of sterilization

Yellow Color indicator for EO sterilization method on the transparent side of the sterile

pouch advises that the sterilization is done and passed.

Warning!

The product is designed for a maximum pressure of 1,800 hPa!

Filling the PEGA® BAG

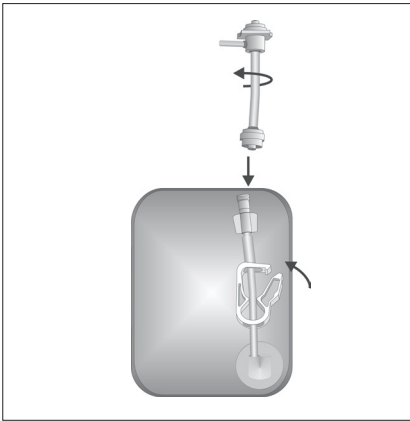
Fig. B: connection PEGA® TUBE

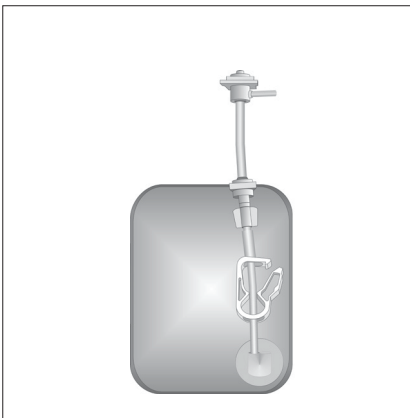
Fig. C: PEGA® BAG with PEGA® TUBE

1. Fill a syringe with the medication solution.
2. Remove the filling cannula if necessary.
3. Connect the syringe to the luer-lock-connector on the PEGA® BAG.
4. Inject the medication solution.
5. Prime the filled PEGA® BAG with the aid of the syringe.
6. Lock the tube of the PEGA® BAG with the tube clamp.
7. Remove the syringe.
8. Use the pinch clamp only for a few minutes to avoid damages of the tube.
9. Lay the filled PEGA® BAG on a flat surface with the Luer-lock-connection facing upwards and the pressure tube of the PEGA® TUBE pointing to the left (**see Fig. B**).
10. Remove the closing caps on the PEGA® TUBE.
11. Connect the luer-lock-connections of the PEGA® BAG and PEGA® TUBE by tightening the PEGA® TUBE with a half turn to the right (**arrow, Fig. B**).
12. Insert the PEGA® TUBE or the PEGA® COMBI LINE into the PEGA® pump according to the instructions for use of your PEGA® pump.

English

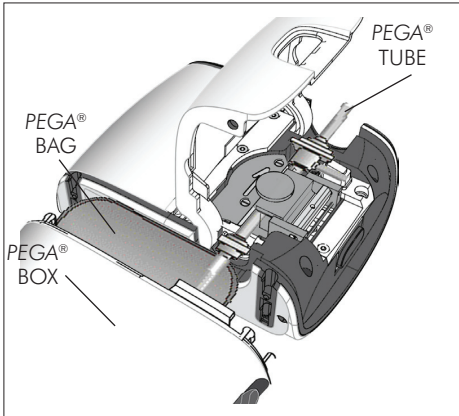
Fig. D: view with infusion pump (Example PEGA® Infuse)

13. Insert the PEGA® BAG into the PEGA® box. The PEGA® TUBE and the tube of the PEGA® BAG must be in line with each other (**Fig. D**).
14. Please make sure that the pinch clamp does not obstruct the flow of drug solution.

Drug Safety

Numbered seals are available on request for the PEGA® BOXes for PEGA® PCA, plus und light pumps.
The PEGA® Infuse BOXes are supplied with key.

Purchase Order

Purchase only complete boxes.

REF.	Bezeichnung	Stück per VE
14050	PEGA® BAG 50 ml	10
14100	PEGA® BAG 110 ml	10
14150	PEGA® BAG 150 ml	10
14300	PEGA® BAG 300 ml	10

Table 1: purchase order

Conformity

Quality and safety of our products are the highest priority at Venner Medical. If a complaint should occur, please contact your responsible medical device consultant or

directly Venner Medical. In case of incidents with the PEGA products, report them to the competent authority in your country (e.g. BfArM in Germany).

Meaning of the symbols on product and transport packaging

 MD	Medical Device	 REF	Article number
	Use by date		Open here
	Single use only. Do not re-use		Keep dry
	Do not re-sterilize		Temperature limit (+5° to +40°C)
	Do not use if package is damaged		Keep away from sunlight
	Caution or Warning		Does not contain phthalates (DEHP)
	Caution, consult instructions for use		Does not contain or no presence of natural rubber latex
	Date of manufacture and country of manufacture		Pyrogen-free
	Batch code		Manufacturer
	Sterilisation with ethylene oxide		
	Single sterile barrier system		
	CE mark with code of notified body - according to MDD 93/42 EEC and MDR		

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