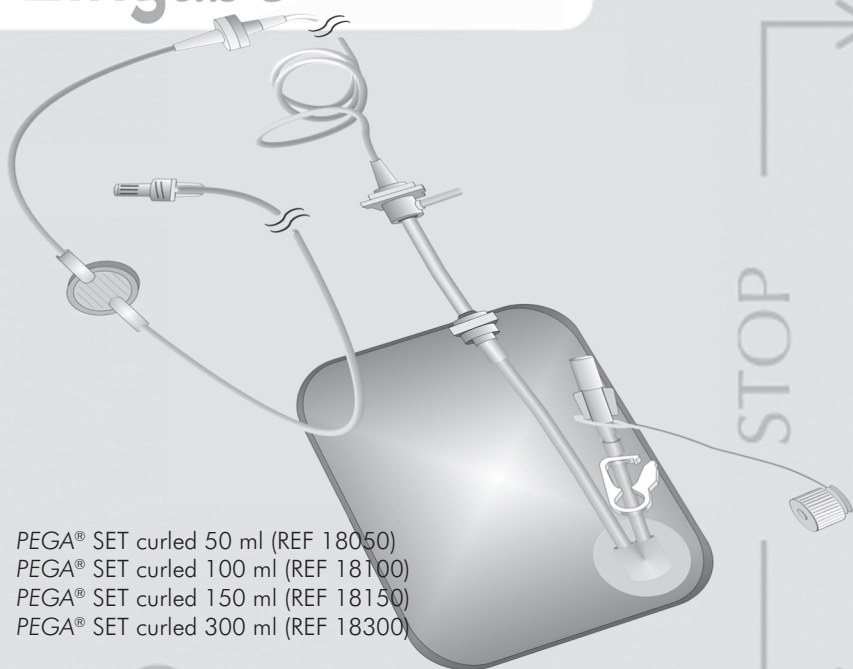
► = Info

◄ = Eingabe

PEGA® SET curled 50 ml (REF 18050)

PEGA® SET curled 100 ml (REF 18100)

PEGA® SET curled 150 ml (REF 18150)

PEGA® SET curled 300 ml (REF 18300)

PEGA® SET curled

CE₀₁₂₃

VENNER

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This page left intentionally blank

Intended Use

PEGA® SET curled is a complete, firmly connected catheter system consisting of a reservoir, a peristaltic tube and an infusion tube with a sterile filter for parenteral administration of drugs in ambulatory and in-patient infusion therapies. PEGA® SET curled consists of a special check valve that prevents back-and freeflow.

PEGA® SET curled is only intended for use with PEGA® Infuse, PEGA® PCA, PEGA® plus and PEGA® light infusion pumps.

Indication for Use

The PEGA® SET curled is suitable for intravenous, subcutaneous, epidural, and intrathecal administration of drugs from a PEGA® infusion pump.


View	No.	Description / Specification
	1	Siphon / Check valve
	2	Connection (patient) / Luer-Lock (male)
	3	Infusion tube L = 2,050 mm, ID = 1 mm, Dead volume PEGA® COMBI LINE $V_0 = 1,600 \mu\text{l}$
	4	Segment of the pump (Silicone) consisting of
	5	Pressure tube (4) and Peristaltic tube (5)
	6	0.2 μm Sterile filter with 0.1 μm PVDF air-out-filter; with embossed device LOT number
	7	Connection (for filling) / Luer-Lock (female)
	8	Tube clamp (PVC)
	9	Medication reservoir 50 ml, 100 ml, 150 ml, 300 ml

Fig. A: PEGA® SET curled

English

Contraindication

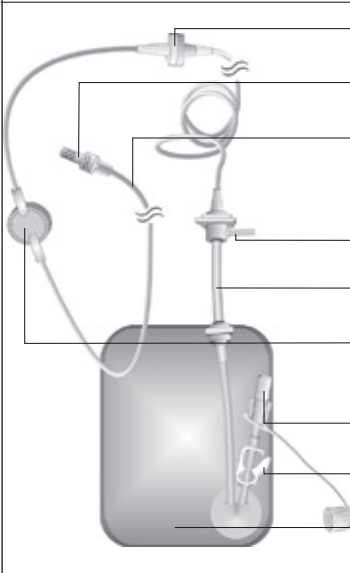Do not use drugs which are not compatible with the material (PVC, PC, PP, PES, PVDF, PMMA, ABS, silicone rubber) of the PEGA® SET curled.

General Safety Instructions

- ◆ Please consult the manual of your PEGA® pump before using a PEGA® SET curled.
- ◆ This disposable item is intended to be used on a patient only once in order to guarantee optimum performance and hygiene.

If the item is used again, even after reconditioning, any liability on the part of Venner Medical (Deutschland) GmbH shall cease to exist.
- ◆ Carry out all of these steps on a clean and low germ concentration surface.
- ◆ Hands must be disinfected previously.

Do not touch open luer-locks (danger of contamination).
- ◆ Use only original PEGA® accessories.
- ◆ In the case of parallel infusions, be sure to connect an additional check valve at the patient end of the infusion tube to prevent different medications mixing in the tube.

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- ◆ PEGA® SETs curled are provided in sterile packages. Do not use any disposable if the package appears damaged or opened.
- ◆ Use PEGA® SETs curled only once and discard the disposable as per local regulations.
- ◆ Only open the vented caps of both luer-lock-connections on the reservoir when filling or priming the system.
- ◆ PEGA® SET curled must not be refilled and not be used longer than 30 days.
- ◆ The pump with the PEGA® SET curled must not be positioned higher than 1 m above the heart of the patient during the therapy.
- ◆ Do not exceed the indicated maximum volume for the PEGA® SET curled.
- ◆ If you want to store a filled PEGA® SET curled for a short time, please remove the vented cap on the luer-lock-connector on the patient's side and replace it with the cap provided.

Confirmation of sterilization

Yellow Color indicator for EO sterilization method on the transparent side of the sterile

pouch advises that the sterilization is done and passed.

Filling the Reservoir

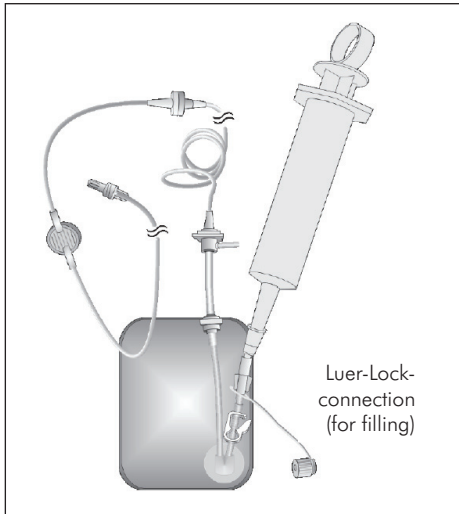
Fig. B: PEGA® SET curled with syringe

Warning!

The product is designed for a maximum pressure of 1,800 hPa!

1. Fill a luer- or a luer-lock-syringe with the medication solution.
2. Remove the filling cannula if necessary.
3. Remove the protective cap from the Luer-Lock connector on the reservoir filling tube and connect the syringe.
4. Inject the medication solution.
5. Prime the filled reservoir with the aid of the syringe.
6. Close the clamp on the reservoir filling tube.
7. Remove the syringe and fit the attached cap.

Preparation for use on the patient

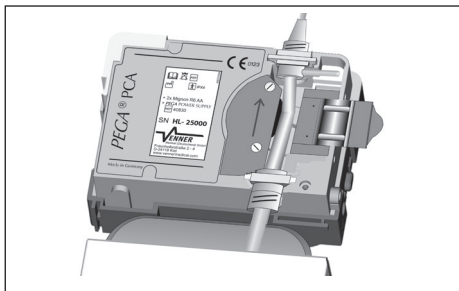
Fig. C: Pump (example PEGA® PCA) with PEGA® SET curled

1. Open the rear pump cover.
2. Open the locking clip.
3. Insert the peristaltic tube into the upper guide of the pump. Ensure that the pressure tube is positioned inside the guides of the coloured pressure sensor.
4. Pull carefully on the Luer-lock-connection of the peristaltic tube so that this engages into the lower guide of the pump.

English

5. If necessary, press the peristaltic tube down with your finger so that is positioned well into the guide.
6. Close the locking clip. The tube is fixed firmly in the peristaltic drive by the pressure block.
7. Place the pump lid on the pump and close it. The clips of the housing (one or two clips, depends on the PEGA® Pump version) must be heard to clip into place.

Caution

The pumps are correctly closed only if the clips are fully engaged. If the clips are not correctly engaged a safe operation of the PEGA® pumps cannot be warranted.

Priming

Before connecting it to the patient, prime the catheter system of the PEGA® SET curled following the instructions in the manual of your PEGA® pump.

Ensure that no air bubbles are left in the tubing or the filter on the patient side (ribbed

side). If this is not the case, flick the filter and tubing lightly with your finger until all the air bubbles are released.

Continue the priming procedure until a drop of medication has formed at the end of the catheter system.

Purchase Order

Purchase only complete boxes.

REF.	Signification	Units per Box
18050	PEGA® SET curled 50 ml	10
18100	PEGA® SET curled 100 ml	10
18150	PEGA® SET curled 150 ml	10
18300	PEGA® SET curled 300 ml	10

Table 1: purchase order

Conformity

Quality and safety of our products are the highest priority at Venner Medical. If a complaint should occur, please contact your responsible medical device consultant or

directly Venner Medical. In case of incidents with the PEGA products, report them to the competent authority in your country (e.g. BfArM in Germany).

Meaning of the symbols on product and transport packaging

 MD	Medical Device	 REF	Article number
	Use by date		Open here
	Single use only. Do not re-use		Keep dry
	Do not re-sterilize		Temperature limit (+5° to +40°C)
	Do not use if package is damaged		Keep away from sunlight
	Caution or Warning		Does not contain phthalates (DEHP)
	Caution, consult instructions for use		Does not contain or no presence of natural rubber latex
	Date of manufacture and country of manufacture		Pyrogen-free
 LOT	Batch code		Manufacturer
 STERILEEO	Sterilisation with ethylene oxide		
 SBS	Single sterile barrier system		
 CE 0123	CE mark with code of notified body - according to MDD 93/42 EEC and MDR		

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